## UDS - Ulnar Deviation Support

EN **HAND ORTHOSIS**

DE **HANDORTHESE**

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IT **ORTESI PER MANO**

NL **HANDORTHESE**

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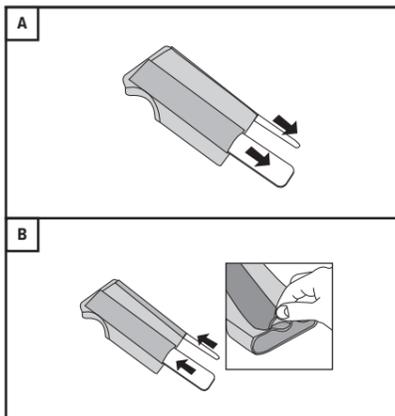
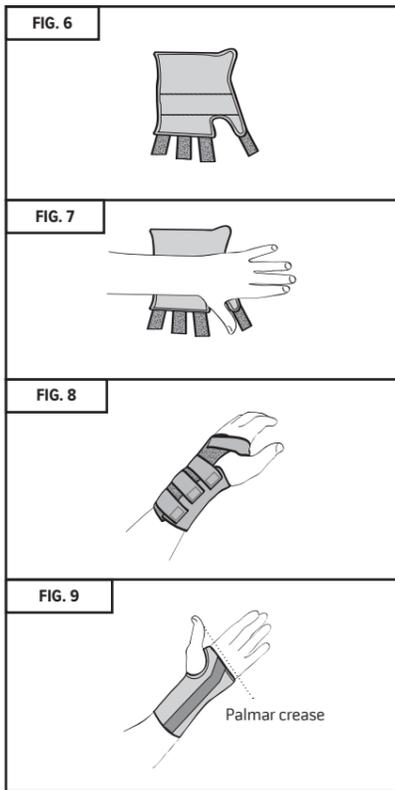
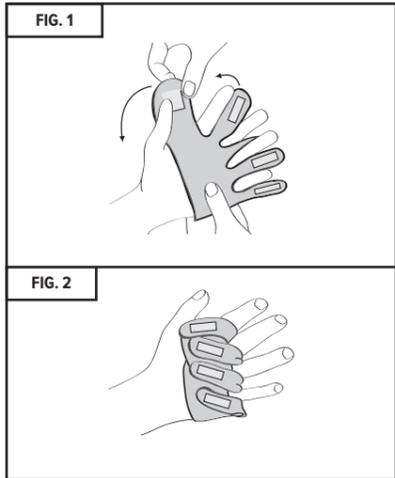
FI **KÄMMENORTTOOSI**

SE **HANDORTOS**

NO **ORTOSE, HÅND**

PL **ORTEZA DŁONI**

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**CE** **EC** **REP** **EMERGO EUROPE**  
Westervortseidsijk 60,  
6827 AT Arnhem  
The Netherlands

©2023 Promedics Orthopaedics Ltd. All Rights Reserved.

UDSIFUB revised 3/23

### ENGLISH

#### Product Description

Helps support the MCP joints and reduce ulnar drift whilst allowing functional use of the fingers.

#### Usage Information (See Fig. 1 & 2)

- Place the wide mid section of the support across the palm with one end passing through the web space. Fasten the hook end to the back of the hand. The support should be just proximal to the MCP joints (Fig. 1).
- Position the first palmer extension piece so it lines up between the index and middle finger (Fig. 1) Bring the extension piece through the index's web space, then around the index finger and pull it diagonally toward the radial edge of the hand. Fasten it in place on the wide support with the hook tab.
- Continue in the same fashion, wrapping the next extension piece around the next finger and attaching it to the previous extension, overlapping each in a radial direction. The last extension piece wraps dorsally around the little finger to attach to the previous extension (Fig. 2).
- Trim the splint and strap end as needed.
- To remove: If tolerable and to save time, unhook only the dorsal hand section and leave the support without unfastening the finger pieces. Then it is pre-set to be applied without major adjustments.

For Paediatric Ulnar Deviation Splint see Fig. 6 - Fig. 9

#### Indications for Use

Provides support for the MCP joints and reduce ulnar drift whilst allowing functional use of the fingers. Manufactured from neoprene to provide warmth and support. Suitable for such conditions as rheumatoid arthritis.

It is recommended that individuals seek guidance from a medical practitioner regarding the most appropriate brace and wearing schedule.

#### Contraindications

- Do not use over open wounds.
- Any known allergy and/or hypersensitivity to any material listed in product composition.

#### Warnings and Precautions

- We recommend the initial fitting of this device be conducted by a suitably qualified healthcare professional who will advise the period of use.
- Follow all instructions to ensure proper performance of the brace.
  - Do not use if liniments, ointments, gels, creams or any other substances have been applied to the affected area.
  - Do not re-use for another patient, doing so risks cross infection and can compromise product integrity.
  - Perform regular skin and circulation checks, especially patients with diabetes, vascular deficiencies and neurological conditions.
  - Should any adverse reactions occur, please cease use and contact your healthcare professional or provider of this product.
  - The durability of the device may be compromised by certain factors, e.g., objects with sharp edges or damage to the hook and loop fasteners. To prevent this the hook and loop fasteners should always be fastened when the device is not being worn or when being washed.

#### Washing and Care Instructions

- Close over all hook/loop straps & remove metal splint (see diagram A).
- Hand wash brace in cool water using mild detergent.
- Rinse and dry flat.
- Do not spin or tumble dry.
- Do not use fabric conditioner at any time.
- Check for any degradation that may affect use.
- Periodically check hook/loop fasteners are operating securely.
- Before using brace again, replace metal splint ensuring splint is correct way round and splint is tucked under edge / 'lip' of pocket. (See diagram B).

#### Product Family Composition

Aluminium, Elastane, Lycra, Nylon, Polyamide, Polyester, Polyurethane, SBR, Spandex, TPU.

#### Storage and Transport Conditions

Store in a cool, dry place out of direct sunlight, in the original packaging.

#### Recycling and Disposal

Packaging and constituent parts should be recycled or safely disposed in accordance with local or national laws.

#### Serious Incident

Report any serious incident to the manufacturer and the competent authority of the EU Member state, or country in which you reside.

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