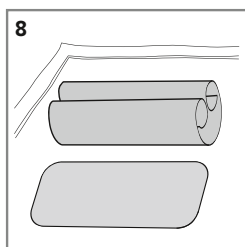
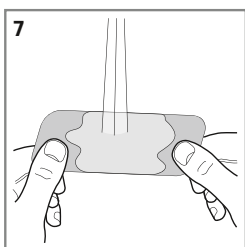
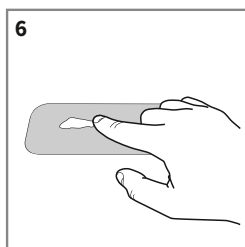
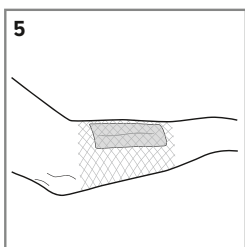
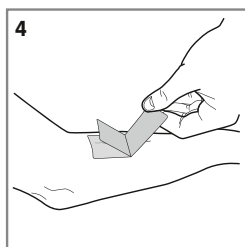
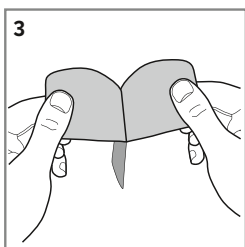
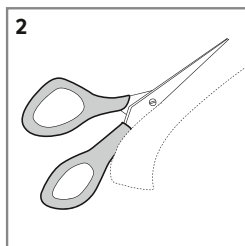
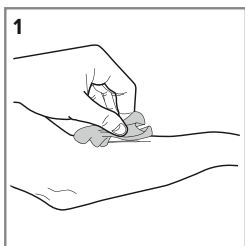
Juzo ScarPad

Silikonauflagen · Silicone sheets

Gebrauchsanweisung · Instructions for use · Mode d'emploi · Istruzioni d'uso · Instruções de utilização · Instrucciones de uso · Gebruiksaanwijzing · Bruksanvisning · Bruksanvisning · Käyttöohje · Bruksanvisning · Οδηγίες χρήσης · Kullanım talimatları · Upute za uporabu · Instrukcja użyci · Инструкция по эксплуатации · Návod na používanie · Navodila za uporabo · تعليمات الاستخدام

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Intended purpose

The recommended treatment of hypertrophic scars and keloids in adults, young people and children is a combination of silicone and compression. Combining the compression garment's consistent surface pressure with silicone can have a positive healing effect on the scar. In addition, functionality and appearance of the scarred area can be improved. Medical silicone allows a moist environment to develop on the scar, meaning that it does not dry out. It keeps the scar soft and has a beneficial effect on the scar tissue. This results in significant pain reduction and improvement of the scar's elasticity.

Notes for use

Your specialist medical retailer will gladly show you the correct way to use the product. The Juzo ScarPad is a self-adhesive silicone sheet for scar treatment. It is available as Juzo ScarPad Light small, Juzo ScarPad Light, Juzo ScarPad Extra and Juzo ScarPad Strong. The product offers a certified UPF 50 UV Protection Factor in accordance with UV Standard 801. The UV protection only applies to those body regions that are covered with the ScarPad. The sheet can be individually adapted and cut to any required shape and size. The Juzo ScarPad can be washed and used repeatedly. It should be worn under compression garments. The Juzo ScarPad should be used as prescribed by your doctor. Gradually increase the usage duration. At the beginning of treatment, use the silicone sheet for three hours every day. If your skin tolerates the silicone sheet well, you can increase the wearing duration by an hour per day, up to 23 hours. The slow increase can help the skin to get used to the effect of the silicone sheet. To ensure the best treatment result, wear the sheet for at least twelve hours every day after getting used to it.

Silicone sheet

1. First of all, thoroughly clean the scar and the surrounding skin. Keep the scar tissue dry and free from oils (Fig. 1).
2. Cut the silicone sheet to the required shape and size. Ensure you're cutting round shapes, if possible, rather than corners (Fig. 2). To guarantee the best result, completely cover the scar with the silicone sheet. Ideally, the silicone sheet should extend beyond the scar by about 2 cm.
3. Remove the protective film from the silicone sheet (Fig. 3).
4. Gently place the self-adhesive side of the silicone sheet on the scar (Fig. 4).
5. Now carefully pull your compression garment over the silicone sheet (Fig. 5).

When you remove the silicone sheet, please make sure you loosen it in several places first, then carefully pull it off.

To care for your skin, use moisturising skin cream **after removing** the silicone sheet. Do not use ointments, creams, gels or lotions in combination with the Juzo ScarPad.

Please note

Silicone sheets are intended for external use. Never dry silicone sheets in the sun or close to heat sources. Store the silicone sheets out of the reach of children. Keep it out of the hands of children. For use only on healed, closed skin. Silicone sheets are manufactured in compliance with the highest quality standards. But if you notice wear and tear on the edges, you can trim them yourself. To do this, stick the silicone sheets on plastic film and trim the edges. For hygiene reasons, this product is intended for one person only.

Washing and care instructions

Juzo ScarPad Cleaning Soap, which is supplied with the product, is a cleaning agent for silicone sheets.

1. Apply a few drops of the Juzo ScarPad Cleaning Soap to the silicone sheet (Fig. 6). Thoroughly clean the silicone sheet.
2. Rinse the sheet under warm, running water (Fig. 7).
3. When the silicone sheet comes in contact with water, it rolls up (Fig. 8). The adhesive properties are completely lost.
4. To dry the (rolled-up) silicone sheet, place it on a dry cloth, with the adhesive side up. When the silicone sheet has dried completely, it unrolls again and its adhesive properties are restored. Only ever use the silicone sheet when it's dry.

Materials used

100 % pure medical grade silicone, elastic layer (80 % polyamide, 20 % spandex), protective backing.

Storage information and shelf life

Store at room temperature in a dry place and protect from direct sunlight. Standard products have a maximum shelf life of 36 months. The shelf life is printed on the box with an hourglass symbol.

Juzo ScarPad Light small and Juzo ScarPad Light are intended to be used for two to four weeks, based on their medical effectiveness. For Juzo ScarPad Extra and Juzo ScarPad Strong, a usage period of four to eight weeks is intended. This period depends on correct handling and normal use of the products.

Indications

Damage to the skin and healing function of the skin with development of hypertrophic scars and keloids after injuries/burns/surgery.

Contraindications

Do not use this product on open wounds or with torn skin. Do not use on irritated skin. Do not use in case of a known intolerance to the component (ultra-pure silicone). If skin intolerances occur during use, please stop using the silicone sheet and consult your doctor. Liability for failure to comply with these contraindications cannot be accepted by SCARPRO N.V..

Side effects

There are no known side effects when this product is used correctly. However, if you notice any adverse changes (e.g. irritation of the skin) during the prescribed use, please consult your doctor or medical retailer immediately. If you know that you are intolerant to one or more of the product's components, please consult your doctor before using the product. If your symptoms worsen during the time you are wearing the product, please consult your doctor immediately. The manufacturer does not accept any liability for damage/injuries resulting from improper use or misuse.

In the event of complaints related to this product, such as damage to the sheet, please contact the medical retailer directly. Only serious incidents that may lead to a major deterioration in the patient's health or to death should be reported to the manufacturer and the relevant authority in the Member State. Serious incidents are defined in Article 2 (65) of Regulation (EU) 2017/745 (MDR).

Disposal

As a rule, the product is to be destroyed via the conventional disposal route after the end of use. Please observe the applicable local/national regulations.

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Vertrieb / Distributor: Julius Zorn GmbH · Juliusplatz 1 · 86551 Aichach · Germany · juzo.com
Berro AG · Austrasse 7 · 9055 Bühler · Suisse