friction brake knee

رَكْبَة مَانَعَة للاحتكاك • knæprotese med friktionsbremse • Reibungsbremsknie
 γόνατο με πέδη τριβής • rodilla con freno de fricción • kitkajarrupolvi
 genou avec frein à friction • ברכ בלם חיכוך • ginocchio con freno ad attrito
 knie met wrijvingsrem • kne med friksjonsbrems • kolano z hamulcem ciernym
 joelho com travão de fricção • joelho com freio de fricção
 коленный сустав с фрикционным тормозом • kolenný kĺb s trecou brzdy
 friktionsbromsande knä • sürtünme frenli diz • 摩擦制动膝盖

college park
 TECHNOLOGY for the HUMAN RACE

user guide

دليل المستخدم • brugervejledning • Handbuch • οδηγός χρήσης
 manual del usuario • käyttöopas • Guide de l'utilisateur
 מדרוך למשתמש • guida per l'utente • gebruikershandleiding
 guia do utilizador • guia do usuário • руководство пользователя
 príručka pre používateľa • användarguide • kullanım kılavuzu • 用户指南

Thank you for using the College Park Guardian Knee!

Your prosthetist will instruct you on the proper use of the knee. If you do not understand the knee or have any questions about using it, please ask your prosthetist.

CARE FOR YOUR COLLEGE PARK GUARDIAN KNEE

If there is no cosmetic cover applied, you may clean the College Park Guardian Knee by wiping it with a soft, lint-free cloth. Never apply harsh chemicals.

CAUTIONS

- Avoid pinching hazards! Do not place fingers near the flexing area of the knee.
- Always report any concerns to your prosthetist immediately, including but not limited to: noise, sudden loss of function, brake release sticking, etc.
- Never attempt to adjust the College Park Guardian Knee yourself. Only a trained professional should make adjustments to the College Park Guardian knee.
- Do not disassemble the knee. Your prosthetist can contact College Park to arrange a repair or replacement.
- Do not expose this product to moisture, i.e. fresh water, corrosive materials, salt water or pH extremes. If the knee encounters moisture, wipe it dry using a lint-free cloth.
- Contaminants such as dirt and the use of lubricants or powder may affect the function of the knee brake and lead to failure.
- Do not use compressed air to clean the knee as it can push dirt inside of the knee.

Failure to follow these technical instructions or use of this product outside the scope of its Limited Warranty may result in injury or damage to the product.

USING THE COLLEGE PARK GUARDIAN KNEE

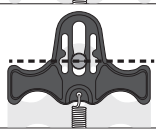
The College Park Guardian knee has two functional modes. Before you begin using the knee, your prosthetist will determine the setting that is most appropriate for you and will activate that mode.

Stance Control Mode - Knee operates normally

Locking Mode - Knee will not flex until the dual function lever is pulled

USING THE MANUAL LOCK FEATURE

If the locking mode has been activated, there will be a dual function lever attached to your prosthesis so you can control when it should be locked or unlocked.

	Locked Position Standing/Walking: The knee will not flex
	Lock Release Position Pull upward on the lever only half-way, without passing the detents. Releasing the lever will return it to the locked position.
	Unlocked Position Deactivates the lock function temporarily: Pull upward on the lever all the way, passing the detents. There will be a “click” sound and feeling. Push down on the lever to return to the locked position.

TO SIT DOWN

1. Pull upward on the dual function lever to allow the knee to flex.
2. If the “toe load” feature is enabled: You must shift your weight forward slightly while also pulling upward on the lever.

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©2022 College Park Industries, Inc. All rights reserved.