

# Select the Product that Best Serves Your Patient's Needs

Comprinet Pro® Anti-Embolism Stockings	JOBST Medical Legwear	JOBST Travel Socks
<p><b>Compression</b> Mild compression (18mmHg)*</p> <p><b>Use</b> Helps reduce the risk of a blood clot and ankle and leg swelling in the immobile (non ambulant) patient</p> <p><b>Patient</b> For bed rest patients (patients that are immobile)</p> <p><b>Styles</b> Unisex style – Knee High, Thigh High</p> <p><b>Colours</b> White</p> <p><b>Sizing</b> Calf measurement plus a leg length</p> <p><b>Other</b> <ul style="list-style-type: none"> <li>• Soft, breathable fabric</li> <li>• Lightweight and comfortable</li> <li>• Inspection window</li> </ul> </p>	<p><b>Compression</b> Moderate to extra firm compression (15-20mmHg, 20-30mmHg, 30-40mmHg)**</p> <p><b>Use</b> Helps manage vascular diseases such as: <ul style="list-style-type: none"> <li>• Varicose veins</li> <li>• Ankle and leg swelling</li> <li>• Chronic venous insufficiency</li> <li>• Venous stasis ulcers</li> <li>• Post-thrombotic syndrome</li> </ul> </p> <p><b>Patient</b> For patients who are active and walking</p> <p><b>Styles</b> Unisex style – Knee High, Thigh High, Pantyhose, Maternity</p> <p><b>Colours</b> Natural, Suntan, Black, Navy, Brown, Beige. NB: Colour dependent on product selected</p> <p><b>Sizing</b> Based on limb circumference</p> <p><b>Other</b> <ul style="list-style-type: none"> <li>• Fashionably appealing</li> <li>• Look just like regular hosiery</li> <li>• Long wearing comfort</li> <li>• Latex free formula</li> <li>• High patient compliance and acceptance</li> </ul> </p>	<p><b>Compression</b> Mild compression (15-20mmHg)**</p> <p><b>Use</b> Helps counteract the effect of inactivity during travel</p> <p><b>Patient</b> For people travelling over a few hours by plane, car, train or bus</p> <p><b>Styles</b> Unisex style – Knee High</p> <p><b>Colours</b> Black or Beige</p> <p><b>Sizing</b> Shoe size or ankle and calf circumference</p> <p><b>Other</b> <ul style="list-style-type: none"> <li>• Soft cotton blend</li> <li>• Excellent breathability</li> <li>• Feels soft on skin</li> <li>• Stylish unisex design</li> <li>• Ideal for business or casual dress</li> </ul> </p>
	<b>See below for selection process.</b>	<b>See below for selection process.</b>

\*Nominal compression for an average ankle size  
\*\*Mean compression for an average ankle size

## Selection Process for JOBST Medical Legwear

### Step 1 Select Required Compression

#### Indications

##### 15-20mmHg

- Tired aching legs
- Minor ankle, leg and foot swelling
- Minor varicose veins during pregnancy
- Long distance travel

##### 20-30mmHg

- Tired aching legs
- Moderate to severe varicose veins during pregnancy
- Post vein surgery/post sclerotherapy
- Moderate tendency of oedema
- Superficial thrombophlebitis
- Helps prevent recurrence of ulcerations

##### 30-40mmHg


- Severe varicose veins
- Lymphatic oedema
- Post vein surgery/post sclerotherapy
- Venous ulcers
- Manages manifestations of PTS, CVI/CVD
- Aids in prevention of thrombotic syndrome

JOBST Medical Legwear should not be worn and is contraindicated for the following conditions:

- Ischemia (eg. Advanced arterial disease) \*
- Uncontrolled congestive heart failure
- Untreated septic phlebitis
- Phlegmasia coerulea dolens

\* As part of a comprehensive assessment, a Doppler Ultrasound is used in the vascular screening process to clarify the vascular aetiology. A hand-held Doppler will assist in determining the degree, if any, of arterial disease. Using the Doppler the Ankle/Brachial Index (ABI) can be calculated.

### Step 2 Select Garment Style

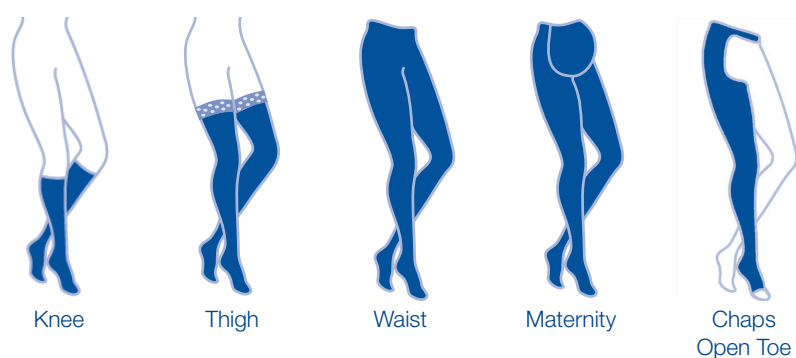


Some considerations when selecting the most appropriate style for patient:

- Indication being treated/managed
- Patient's ability to manage and tolerate hosiery
- Patient's skin condition
- Shape, size and function of the limb
- Patient's lifestyle, mobility and age
- Open toe variants only available in JOBST Relief
- Maternity Style only available in JOBST UltraSheer

### Step 3 Measuring For JOBST Medical Legwear

#### Provide the following measurements:

- Knee High – b & c
- Thigh High – b, c & g
- Waist High – b, g & k

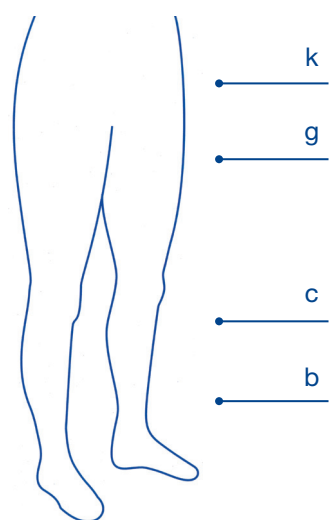

##### k – Hip

Circumference at large dimension around buttocks.

##### g – Thigh

Circumference at largest dimension.

##### c – Calf

Circumference at largest dimension.

##### b – Ankle

Circumference at smallest dimension.

### Step 4 Select the most appropriate JOBST Ready to Wear Product



#### JOBST UltraSheer

- Beautifully sheer
- Fashionable
- Light weight and comfortable
- Durable and long lasting
- Available in Knee High, Thigh High, Waist High and Maternity variants



#### JOBST forMen

- Fine quality dress socks
- Long-lasting, lightweight and stylish
- Soft, skin friendly fabric
- High durability and patient comfort
- Available in Knee High and Thigh High variants



#### JOBST Relief

- Classical therapeutic stockings
- High durability
- Easy to don
- Increased patient comfort
- Available in Knee High, Thigh High, Waist High and Chaps – Open and Closed toe variants also available



#### JOBST Travel Socks

- Soft cotton blend for additional comfort and breathability
- Specifically designed to counteract swelling and discomfort during prolonged inactive periods of travel
- Clinically proven gradient compression
- Soft cotton blend
- Available in Knee High

### Step 5 Complete JOBST Medical Legwear Prescription Pad

#### Important to Note

- It is important that you discuss the wear and care of JOBST Medical Legwear with your patient. All information relating to this is contained within the product.
- If prescribing your patient to wear their compression

garments on a daily basis, it is important to recommend they have two pairs to ensure they are able to appropriately care for their garments.

