

FS3000 FOOT DROP ORTHOTIC KIT, SIZE MEDIUM (FS3000-KT-M)

(SHOE SIZE 8 US -> 13 US Men)

ASSEMBLY INSTRUCTIONS



SUGGESTED USEFULL TOOLS

- -POWERED DRILL (VARIABLE SPEED)
- -DRILL BITS 7/64", 1/4" (2.7MM, 6.35MM)
- -MEASURING CALIPER 300MM OR MEASURING TAPE
- -ALLEN KEY 1.5MM (INCLUDED)
- -METAL HAND SAW
- -BENCH VISE WITH SOFT PAD JAW (PLASTIC OR LEATHER)
- -TRAUTMAN SANDER AND CARVER
- -PROPANE BLOW TORCH OR HEAT GUN (IF THERMOFORMING REQUIRED)
- -SNAP OFF BLADE KNIFE (OLFA, X-ACTO)
- -CUTTER PLIER
- -PROTECTIVE SAFETY GEAR: SAFETY GOGGLES, GLOVES, DUST MASK
- & ALL OTHER PROTECTION REQUIRED BY YOUR HEALTH & SAFETY WORK POLICY)



WARNING: NEVER ASSEMBLE, MODIFY, REPAIR, UNSASSEMBLE, CORRECT, CUT, FOLD, PIERCE, SAND, HEAT WHEN A PATIENT WEARS THE ORTHOSIS. ALWAYS REMOVE THE ORTHOSIS FROM THE PATIENT AND TAKE OFF THE SHOE BEFORE TRYING TO PERFORM ANY WORK.

Always use safe work methods and follow your local health and safety policy. Inspect all tools for any damage prior to each use and discard immediately if damage is suspected. Always read and follow the operating and safety instructions in the power tool owner's manual and the instructions. Always wear safety goggles to protect your eyes, gloves to protect your hands, and wear other appropriate safety items for protection.

Keep bystanders, children and visitors away while operating a power tool. Stay alert, watch what you are doing and use common sense when operating a power tool. Do not use tools while tired or under the influence of drugs, alcohol or medication. Do not wear any loose clothing or jewelry that can catch in the spinning rotating tools. Tie back long hair or wear appropriate hair protection to prevent hair from getting caught in moving parts of equipment.

For electric powered tools, check the handle and body casing of the tool for cracks or other damage. Inspect cords for defects: check the power cord for cracking, fraying, and other signs of wear or faults in the cord insulation. Do not use electric tools in wet conditions or damp locations. Always unplug the power tool before changing drill bits. A lways inspect the drill bits prior to use, and discard immediately if damage is suspected. Always drill straight and SLOWLY. Make sure your hands or any body parts stay clear from the backside of the material being drilled. Do not use a side-to-side movement when drilling. Always maintain a firm footing and control of the tool.

Blowtorch: When using a blowtorch of any kind, you should always refer to the manufacturer's instructions before operating, lighting or using. Make sure you put your Personal Protective Equipment on before turning the blowtorch on. Review instructions and cautions periodically to maintain awareness. Always use in a well-ventilated area. Keep torch out of reach of anyone who has not read instructions. Do not point torch towards face, other persons or flammable objects. Never attempt to use torch as a cigarette lighter. Disconnect cylinder when not in use. During heating, avoid holding it horizontally or upside down and try to maintain its position as close to vertical as possible. Never attempt to modify the torch construction and never use unapproved accessories or fuels.

Treat the torch as you would with any fine tool or instrument. Do not use a leaking, damaged or malfunctioning torch when brazing, always wear safety goggles. Work only in well-ventilated . Always wear protective gloves and use proper tools to handle hot work. Be aware that the tip of the torch can get extremely hot during use. Take precautions to protect yourself and others from accidental burns. Never use the torch on or near combustible material. Always make certain the torch is placed on a level surface when connected to the fuel cylinder to reduce the risk of accidental tip over. Be sure the torch is not pointed in a direction that could cause nearby objects to ignite when torch is set down. Always have a fire extinguisher and a bucket of water near the torch and work area. Do not place them in the flame path or in a location where you could not reach them should a fire break out.


Metal hand saw:


When using a hand saw, protect your face and eyes, you must always wear a safety mask and eye protection. Ispect it thoroughly for any signs of damage. If the handle is loose, or the blade is bent or broken, or if there are missing blade teeth, do not use the hand saw. Never test the sharpness of a hand saw with your hand or any other part of your body. Doing so may result in cuts and injuries. To prevent buckling and rebound of the hand saw, you must always start off slowly. Use long, even strokes to start cutting the material. To avoid serious injuries, it is crucial to hold the stock firmly in place. Use a vise or a clamp for this purpose.


Exercise caution if you need to use sharp cutting tools or knife.


US Patent No: 8529484 - CA Patent No 2,692,534 - Patents pending in European Countries
Turbomed Orthotics and Turbomed Orthotics Logo are registered trademarks of Turbomed Orthotics Inc.


	PARTS LIST - ORTHOTIC KIT, SIZE MEDIUM (FS3000-KT-M)											
Item	Qty	PRODUCT CODE	DESCRIPTION									
Α	1	FS3000-Y00-ML	MAIN POSTERIOR SECTION «Y» SHAPE, SIZE MEDIUM-LARGE									
В	1	FS3000-U00-ML	FRONT SHOE SECTION «U» SHAPE, SIZE MEDIUM-LARGE									
С	1	FS3000-W00-ML	REAR HEEL SECTION SUPPORT, SIZE MEDIUM-LARGE									
D	5	FS3000-L00-ML	LACE CLIP ATTACHMENT, SIZE MEDIUM-LARGE									
Е	1	FS3000-C00-ML	CALF SECTION, SIZE MEDIUM-LARGE									
F	2	FS3000-B00-M	ALUMINUM JUNCTION BLOC, SIZE MEDIUM (38mm long.)									
G	14	FS3000-SET-12	M3X12 SET SCREW 18-8 STAINLESS STEEL, 12MM LG									
Н	1	FS3000-K00-SML	COMPLETE STRAP & CUSHIONS KIT, SIZE SMALL-MEDIUM-LARGE									
Ī	20	FS3000-ZZZ-001	TIE-WRAP BLACK 18 LBS/4"LONG									
J	1	FS3000-ZZZ-002	HIGH-TORQUE HEX-L-KEY 1.5MM FOR M3 SET SCREW									

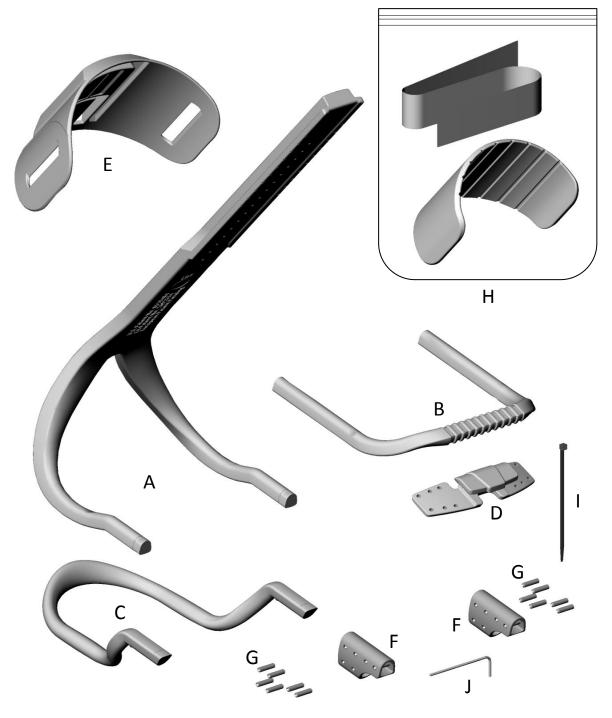


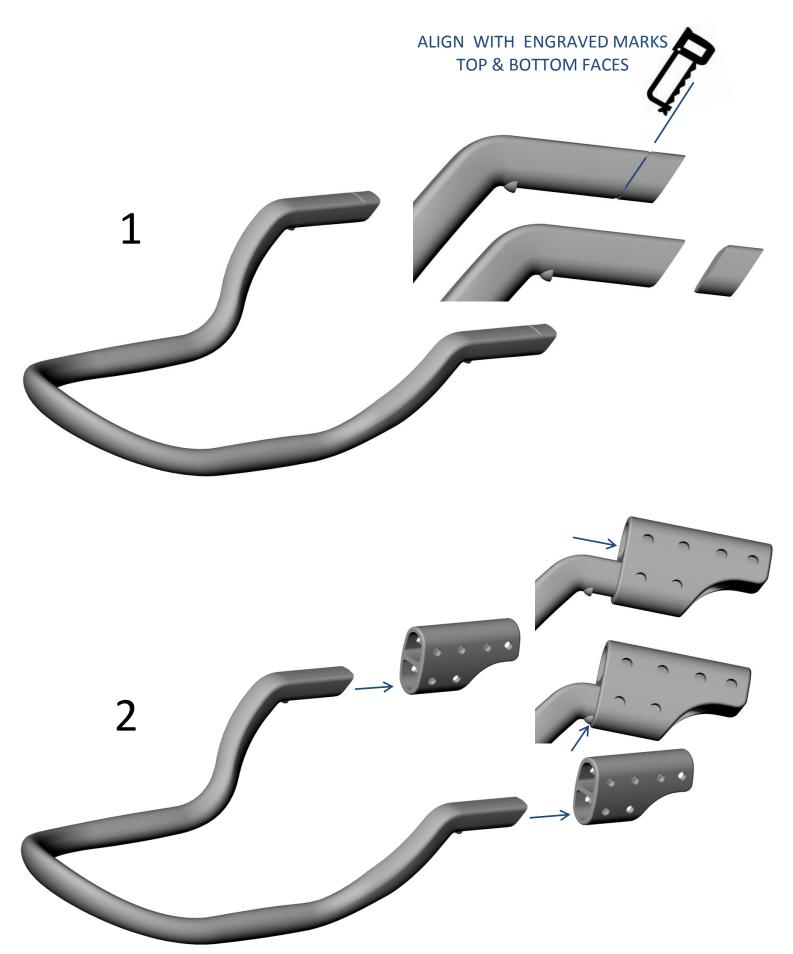


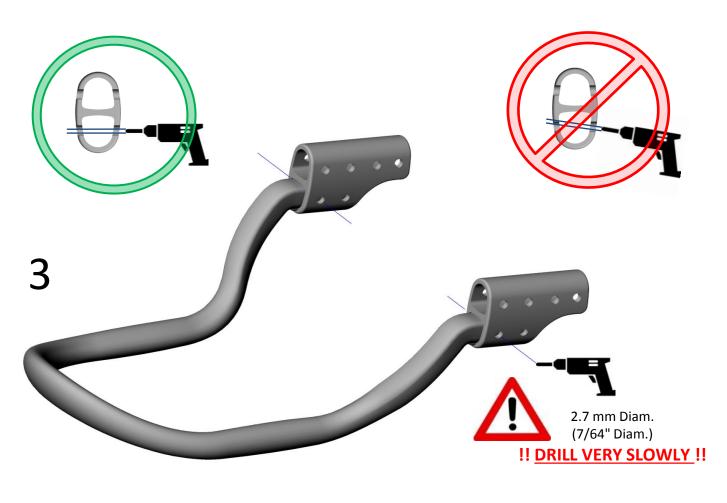


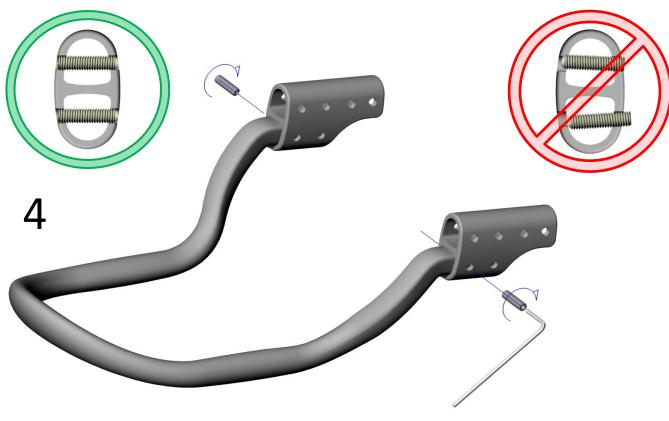


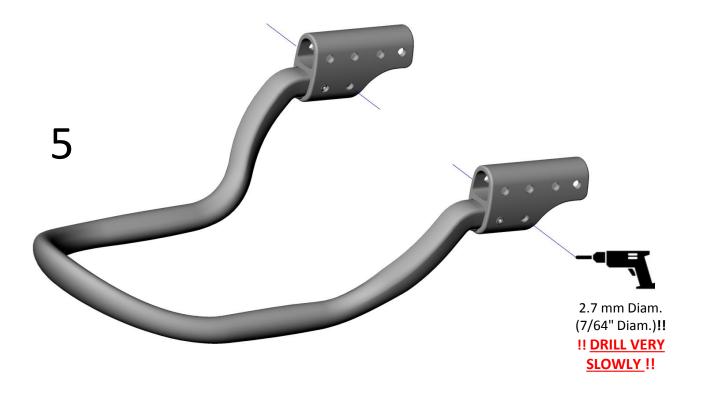


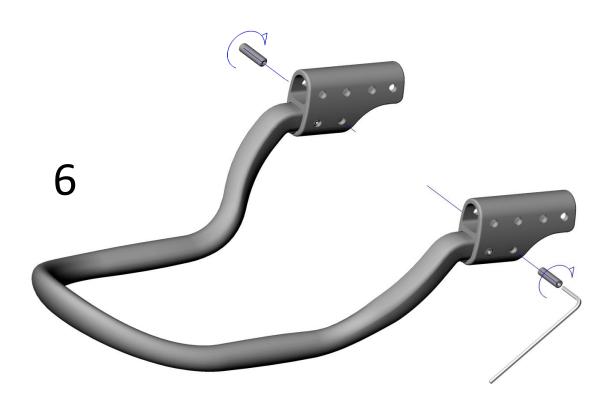


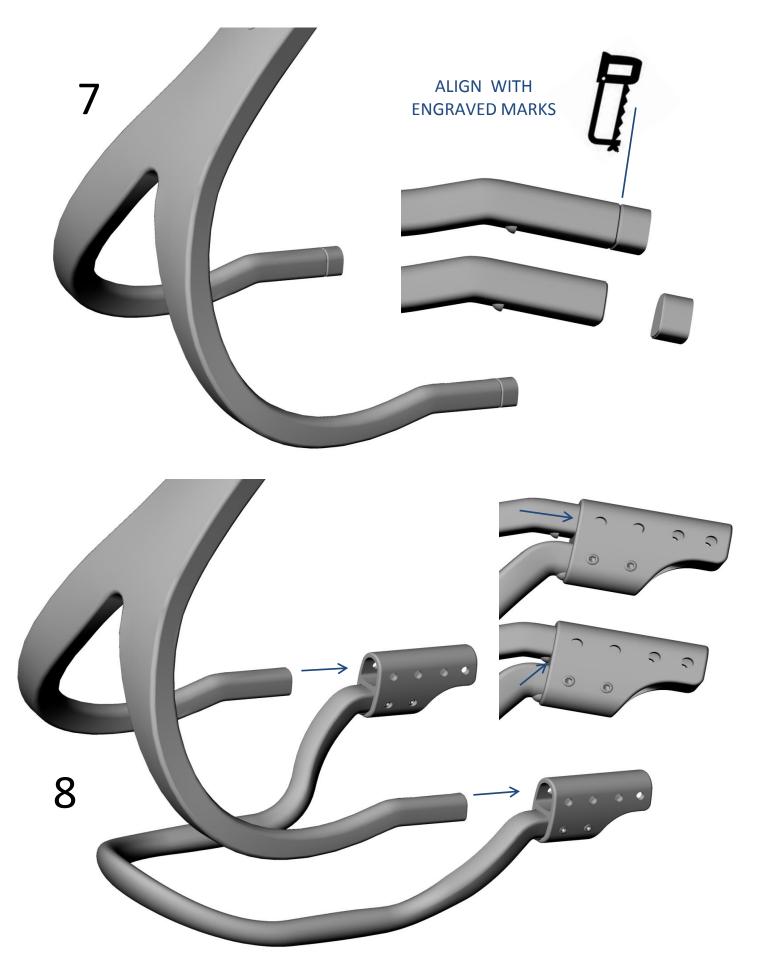

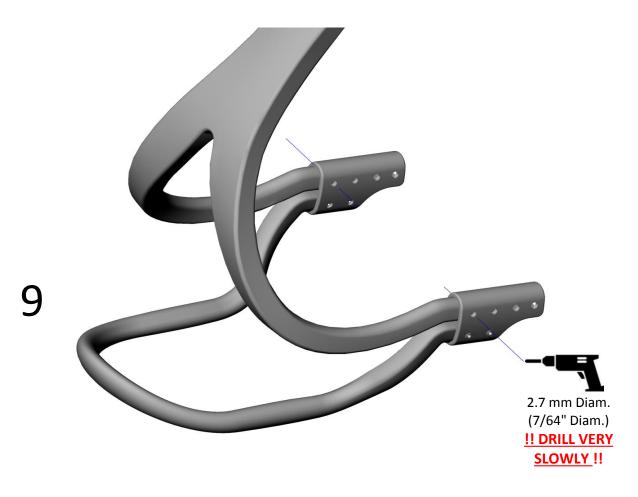

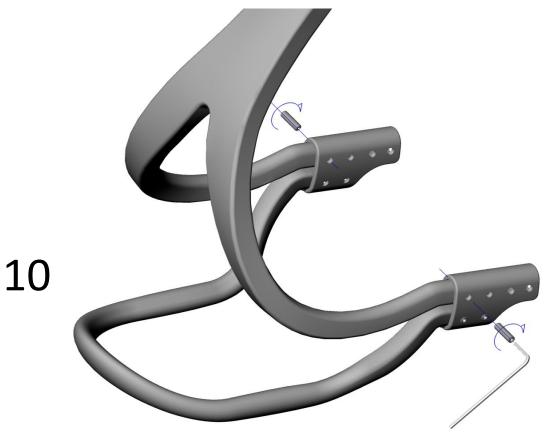


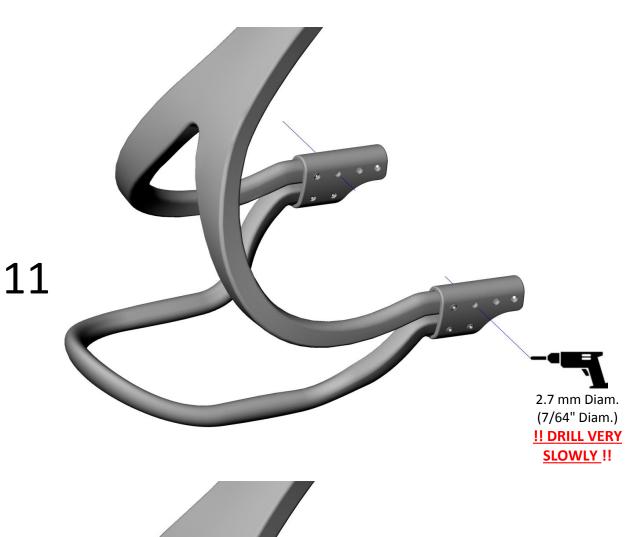


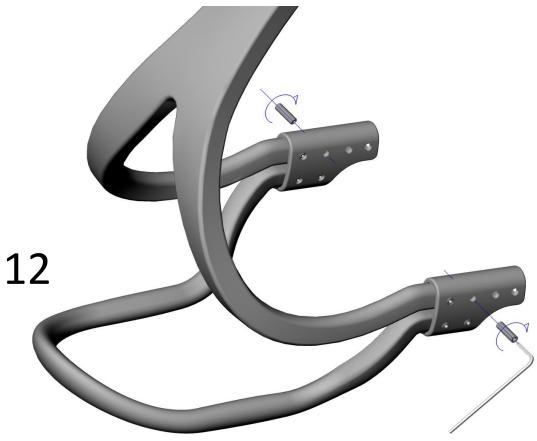



	FS3000 MEDIUM KIT - LENGTH ADJUSTEMENT CHART														
SHOE SIZE	US-M	8	8.5	9	9.5	10	10.5	11	11.5	12	12.5	13			
SHOE SIZE	US-W	9	9.5	10	10.5	11	////	////	////	////	////	////			
FINAL LENGTH	L (mm)	199	203	205	208	211	214	217	220	223	226	230			
CUTTING REF	Χ¬	25	29	31	34	37	40	43	46	50	53	57			


"X"ACCURATE FOR W=96mm(ORIGINAL BRACE & ASSESSMENT TOOL)
ADD 2-4 mm TO ENLARGE BRACE WIDTH ("W")
SUBSTRACT 2-4 mm TO REDUCE BRACE WIDTH ("W")

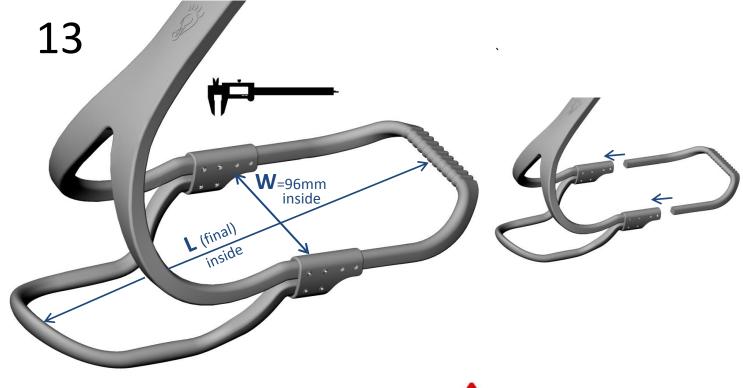

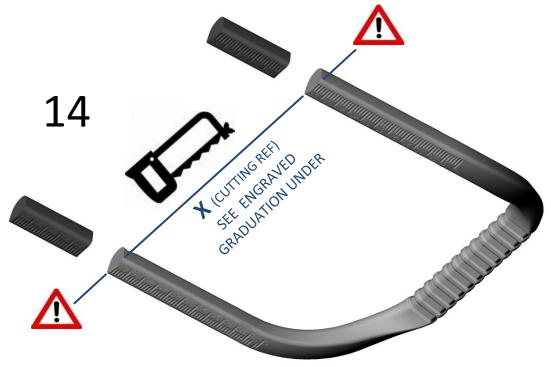

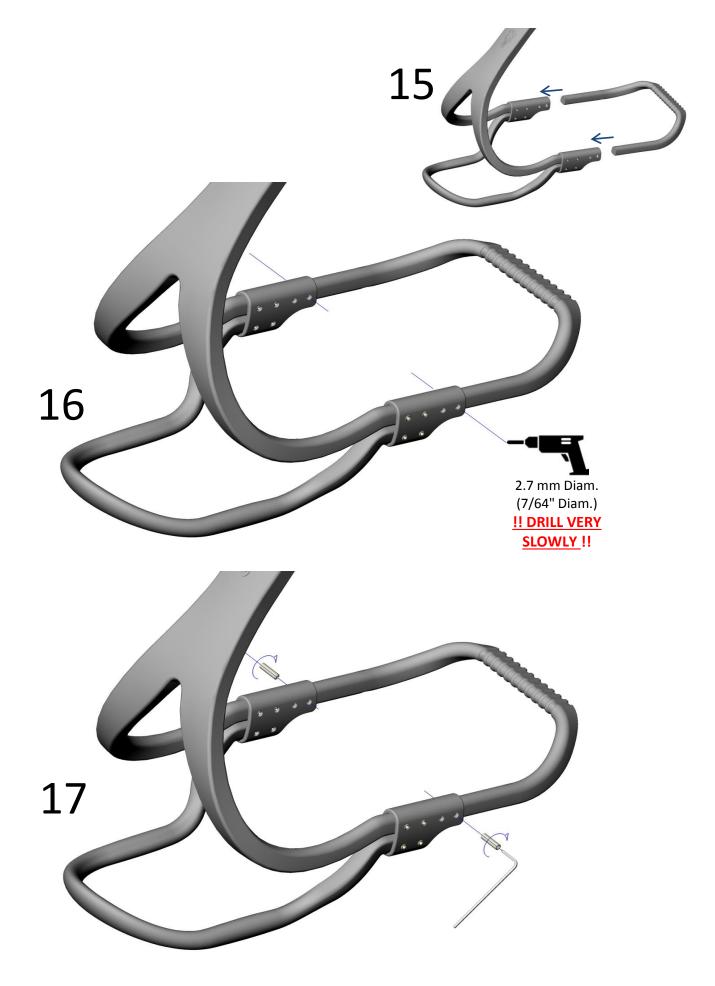


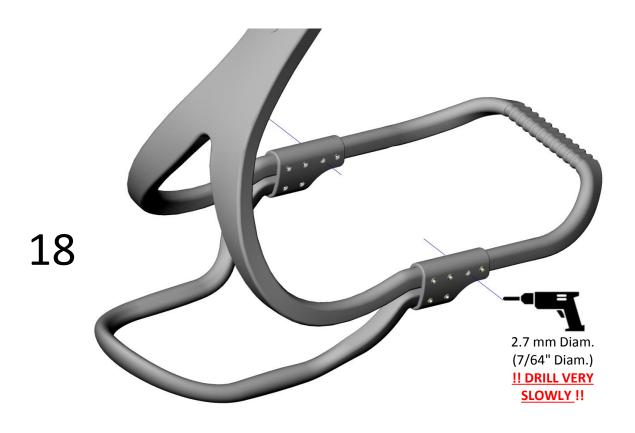


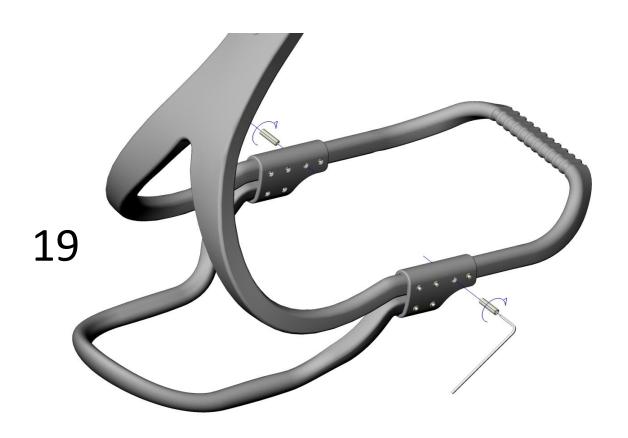



	FS3000 MEDIUM KIT - CALF SUPPORT ADJUSTEMENT CHART														
SHOE SIZE	US-M	8	8.5	9	9.5	10	10.5	11	11.5	12	12.5	13			
SHOE SIZE	US-W	9	9.5	10	10.5	11	////	////	////	////	////	////			
CALF HEIGHT	H (mm)	260	265	265	270	270	275	275	280	280	286	286			

****SUGGESTED HEIGHT (Y) PROPORTIONAL TO SHOE SIZE - WE SUGGEST TO TAKE MEASUREMENTS ON PATIENT ****

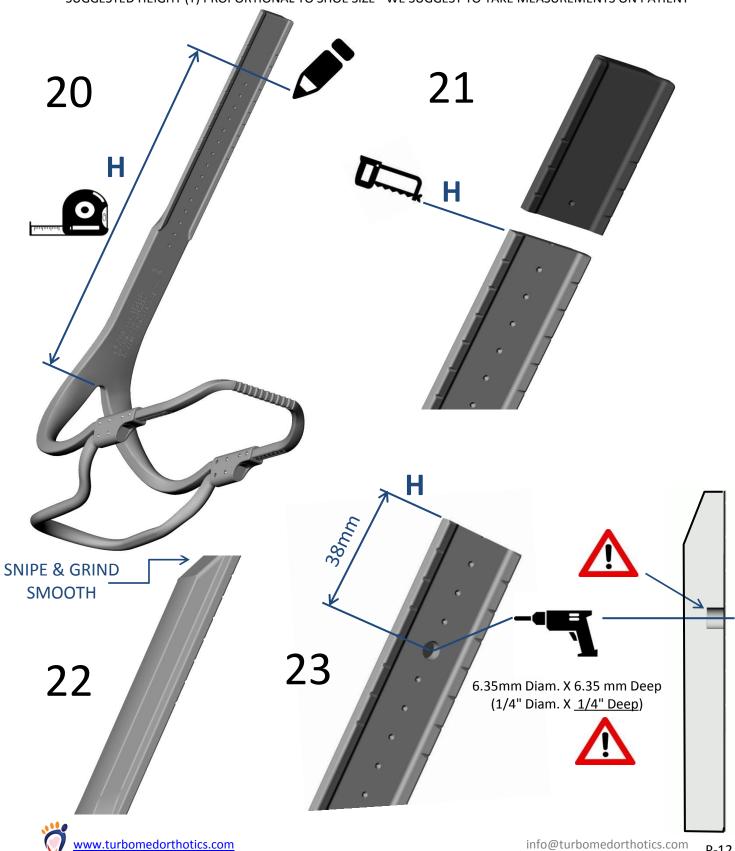

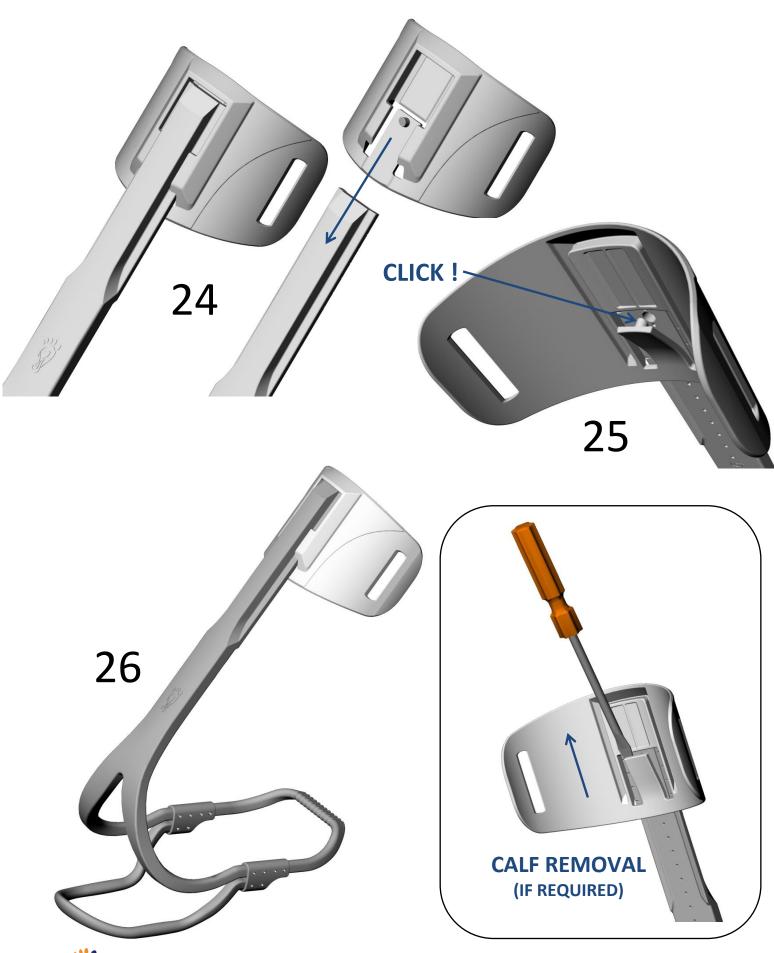

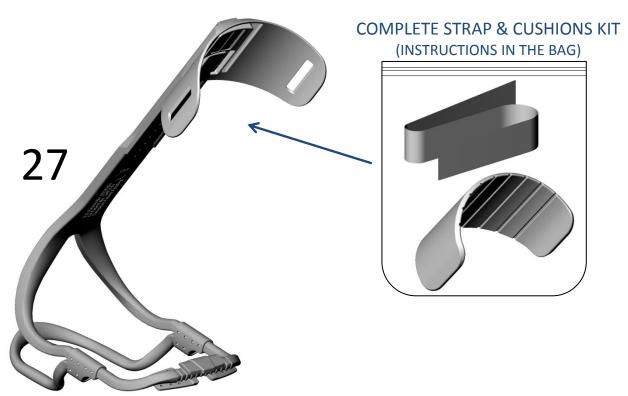

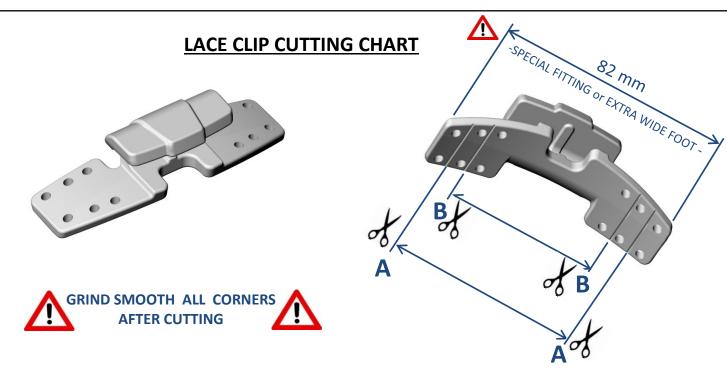

FOR FINAL INSTALLATION ON FOOTWEAR , SEE LACE CLIP FITTING INSTRUCTION DOCUMENT

	FS3000 MEDIUM KIT - LACE CLIP CUTTING CHART														
SHOE SIZE	US-M	8	8.5	9	9.5	10	10.5	11	11.5	12	12.5	13			
SHOE SIZE	US-W	9	9.5	10	10.5	11	////	////	////	////	////	////			
CUTTING POSITION		В	В	В	В	В	В	В	В	В	В	Α			



****SUGGESTED WIDTH TO BE VALIDATED ON FOOTWEAR****

DO NOT CUT UNTIL FINAL FITTING ON FOOTWEAR

ANNEXE - KEY DIMENSION SUMMARY


	FS3000 MEDIUM KIT - SUMMARY ASSEMBLY DIMENSIONS														
SHOE SIZE	US-M	8	8.5	9	9.5	10	10.5	11	11.5	12	12.5	13			
SHOE SIZE	US-W	9	9.5	10	10.5	11	////	////	////	////	////	////			
FINAL LENGTH	L (mm)	199	203	205	208	211	214	217	220	223	226	230			
CALF HEIGHT	H (mm)	260	265	265	270	270	275	275	280	280	286	286			
WIDTH	W (mm)	86	88	88	90	90	94	94	96	96	96	98			







ADD OR SUBSTRACT 2-4mm AT INITIAL LENGTH TO ACHIEVE FINAL FIT LENGTH "L"

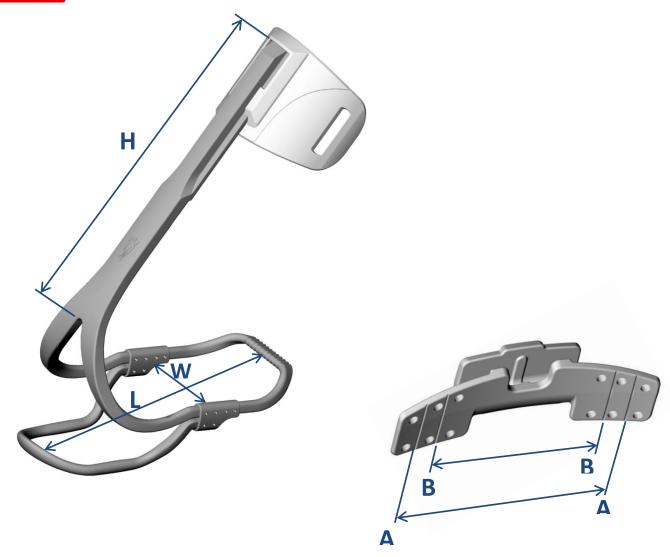

	FS3000 MEDIUM KIT - LACE CLIP CUTTING CHART														
SHOE SIZE	US-M	8	8.5	9	9.5	10	10.5	11	11.5	12	12.5	13			
SHOE SIZE	US-W	9	9.5	10	10.5	11	////	////	////	////	////	////			
CUTTING POSITION		В	В	В	В	В	В	В	В	В	В	Α			



****SUGGESTED WIDTH TO BE VALIDATED ON FOOTWEAR***

DO NOT CUT UNTIL FINAL FITTING ON FOOTWEAR



T: 001-418-563-8675